

# Surface disinfection

## Quick & Clean S-Wipes

Ready to use, alcohol-soaked disinfection wipes for application with dispenser bucket



### Product description

Quick & Clean S-Wipes are ready to use alcohol-soaked disinfection wipes in a refill bag. They are applied with a dispenser bucket which is available separately. The wipes are soaked with PLIWA Lemon Fresh AF (containing alcohol and quarternary ammonium compounds) and can be used directly after insertion into the dispenser bucket. The product provides a universal and time-saving application.

### Areas of application

For rapid disinfection of alcohol-resistant surfaces of non-invasive, non-critical medical devices and medical equipment.

Suitable for:

- Surfaces close to patients, dental chairs, examination couches, operating tables
- Areas in medical and dental practices which are used for medical purposes
- Rapid disinfection between surgeries and treatments
- Clothes like OR shoes, OR aprons, X-ray-aprons
- Alcohol-resistant surfaces of medical equipment

Not tested for food and veterinary areas. Not suitable for disinfection of skin and hands. Not suitable for disinfection of invasive medical devices. Do not use for disinfection of semi-critical and critical medical devices.

### Special advice

**Do not reuse the product. If the disinfectant wipe is reused, the indicated effectiveness is no longer ensured.**

**Do not use the product if it is damaged or soiled.**

**Application only by qualified personnel.**

**Report serious incidents related to this product to the manufacturer and the responsible authority.**

### Application

Put on protective gloves, face mask and eye protection prior to disinfection. Provide adequate ventilation.

- 1) In case of contaminated surfaces: Remove visible contaminations with a disinfection wipe and discard it.
- 2) Disinfect surface with a fresh disinfection wipe. Wipe the surface applying mild pressure (wet wiping). Keep the surface completely moist during contact time.
- 3) After contact time has elapsed: Remove residues with a clean, dry, disposable cloth or allow to dry.

### Shelf life after opening

Shelf life of opened disinfection wipes in dispenser bucket: 28 days from day of opening if the lid is closed properly and tightly every time a wipe is taken out.

The indicated shelf life after opening can only be guaranteed when using the dispenser bucket tested by the manufacturer.

### Summarized efficacy spectrum

Efficacy under dirty conditions (product ready to use, at room temperature)	Contact time
Bactericidal/Yeasticidal (VAH standard methods)	5 min
Bactericidal/Yeasticidal (EN13727, EN 13624, EN 16615)	5 min
Mycobactericidal/TB (EN 14348)	
Limited spectrum of virucidal activity (EN 14476, EN 16777)	30 s
Virucidal activity against enveloped viruses (EN 14476)	

See page 2 for a detailed efficacy spectrum, material compatibility and advice on filling of the dispenser bucket.



# Quick & Clean S-Wipes

Ready to use, alcohol-soaked disinfection wipes for application with dispenser bucket

## Detailed spectrum of efficacy

Efficacy (product ready to use, at room-temperature)	Standard	Contact time
<b>Bacteria</b>		
<i>S. aureus</i> , <i>E. hirae</i> , <i>P. aeruginosa</i> , <i>E. coli</i> , MRSA (dirty conditions)	EN 13727	30 s
<i>S. aureus</i> , <i>E. hirae</i> , <i>P. aeruginosa</i> (dirty conditions)	EN 16615	5 min
<i>S. aureus</i> , <i>E. hirae</i> , <i>P. aeruginosa</i> (dirty conditions)	VAH standard methods*	5 min
<b>Mycobacteria/TB</b>		
<i>M. terrae</i> , <i>M. avium</i> (dirty conditions)	EN 14348	1 min
<b>Yeasts</b>		
<i>C. albicans</i> (dirty conditions)	EN 13624	30 s
<i>C. albicans</i> (dirty conditions)	EN 16615	5 min
<i>C. albicans</i> (dirty conditions)	VAH standard methods*	5 min
<b>Viruses</b>		
<b>Enveloped viruses</b>		
MVA Modified Vacciniavirus Ankara (dirty conditions)	EN 14476	30 s
BVDV (dirty conditions)		
<b>Non-enveloped viruses</b>		
Adenovirus type 5 (dirty conditions)	EN 14476	5 min
MNV murine Norovirus (dirty conditions)		
Human Rotavirus (clean conditions)		
MNV murine Norovirus (dirty conditions)	EN 16777	
Adenovirus type 5 (dirty conditions)		

\*standard methods of the VAH (Association of Applied Hygiene) for surface disinfection by wiping (quantitative carrier test with mechanical action, practical conditions).

## Material compatibility

Metal	Aluminium, stainless steel, copper, brass
Plastics	PSU, PVC, silicone
Rubber	EPDM, latex
Advice	Upon compliance to the intended use and the indicated contact time, no material damage is to be expected with the above mentioned materials. Avoid contact of longer than 48 hours with linoleum flooring. Avoid contact with acrylates. May cause stress corrosion of acrylic glass. Test material compatibility for polyacrylates, polycarbonates as well as not mentioned materials before use.

## Filling of the dispenser bucket

Comply with the dispenser bucket manufacturer's recommendations for reprocessing as well as national requirements concerning the reprocessing of dispenser buckets. Put on protective gloves, face mask and eye protection. Provide adequate ventilation.

- 1) Record product data (provided on the refill bag) on the additional label and attach it to the dispenser bucket.
- 2) Put the refill bag into the prepared dispenser bucket.
- 3) Cut open the refill bag at the top, insert the wipe through the opening of the lid and close the bucket thoroughly. The product is now ready for use.

## Advice

Avoid contact with other disinfectants. Do not mix with other disinfectants. Do not dilute.

Apply on smooth surfaces only. On coarse surfaces lint formation cannot be excluded.

Do not dispose of Quick & Clean S-Wipes down the toilet.

### Further information on cleaning and disinfection of surfaces and medical devices:

Recommendation of the Commission for Hospital Hygiene and Infection Prevention at the Robert-Koch Institute (RKI) entitled „Hygiene Requirements for Cleaning and Disinfecting Surfaces“. Available online.

## Storage advice

Store the product between 5 and 25°C. Protect from heat and sunlight. Store in an upright position and tightly closed in a well-ventilated place. Keep cool. Keep away from sources of ignition – No smoking. Take precautionary measures against static discharge. Keep away from food, drink, and animal feedingstuffs. Keep locked up and out of the reach of children.

## Identification, certificates

CE 0123 MD Ila

## Composition

100 g of soaking solution contain:

33.66 g ethanol, 15.84 g 2-propanol, 0.05 g didecyldimethylammonium chloride 70%, parfum\*2 (not applicable for parfum-free)

## Chemical/physical parameters

appearance: clear and colourless  
density (20°C): 0.900 – 0.920  
flash point (EN 22719) 22.75 + 0.5 °C

\*2 Please note that parfum may cause allergies.



# Quick & Clean S-Wipes

Ready to use, alcohol-soaked disinfection wipes for application with dispenser bucket

## Expert opinions & further information

Quick & Clean S-Wipes have been assessed by independent laboratories. The shelf life after opening of 28 days was confirmed in an expert opinion.

To exclude adverse effects on the skin upon unintentional contact with the product, a dermatological survey has been carried out with the soaking solution by the company Dermatest.

For further details concerning expert opinions, information on products and answers to specific questions contact us by e-mail (info@pliwa.de) or telephone (0049 5661 73170).

PLIWA Hygiene GmbH has a manufacturer's authorisation as per §13 German medicines act as well as a GMP- and an EN ISO 13485 certificate.

## Labelling according to Regulation (EC) No 1272/2008 (CLP)

For information on labelling according to Regulation (EC) No 1272/2008 (CLP) see safety datasheet and label.

## Information for order

Product	Item no.
<b>Lemon Fresh</b>	
70 wipes in refill bag	10-452
90 wipes in refill bag	10-453
<b>Without parfum</b>	
70 wipes in refill bag	10-552
90 wipes in refill bag	10-553
<b>Accessories</b>	
Dispenser bucket	93308
	93309

size of wipe: 300 x 300 mm

